

Matrx[™] VIP 3000 Calibrated Vaporizer

Operation & Maintenance Manual

This page intentionally left blank

Contents

			Page
1.0		oduction	
		Warning and Cautions	
		User Responsibility	
	1.3	Servicing	3
2.0	Des	scription	4
	2.1	General	4
	2.2	Dial Control	5
3.0	Spe	ecifications	5
		Calibration	
		Resistance to Gas Flow	
	3.3	Liquid Capacity	5
		Weight and Dimensions	
		Temperature Range	
4.0	Per	formance	6
		Performance Curves	
		Effects of Variables	
		4.2.1 Anesthetic Consumption	7
		4.2.2 Temperature	
		4.2.3 Barometric Pressure	
		4.2.4 Back Pressure (Steady)	
		4.2.5 Back Pressure (Fluctuating)	
		4.2.6 Carrier Gas Composition	
		4.2.7 Time Out of Service	
		4.2.8 Other Variables	

Principle of Operation	
5.1 Vaporizer Sump and Valve Assembly	10
Installation	
6.1 Mounting the Vaporizer	
Operating Instructions	
7.2.1 General	
7.2.2 Screw Cap Filler	
7.2.3 Keyed Filler	
Checking the Calibration	
8.1 General	
8.2 Analytical Techniques	
Maintenance	
9.4 Repairs	
0 Warranty	
	5.1 Vaporizer Sump and Valve Assembly Installation

1.0 Introduction 1.1 Warning and Cautions

A number of Warnings , Cautions , and Notes are used throughout this manual. These draw attention to possible hazards and adverse conditions that may occur if the information and instructions provided are not strictly observed.



Warnings are used to draw attention to a condition that can endanger either the patient or the operator.



Cautions are used to draw attention to a condition that can result in damage to the equipment.



Notes are used to give conditional information to clarify the instruction.

Special attention must be made to each Warning and Caution as it appears in the manual.

- 1. Federal law restricts this device to sale by or use by a veterinarian.
- The Matrx VIP 3000 calibrated vaporizer is designed for use with one anesthetic agent only, which is that named on the vaporizer. Incorrect dosage may result if the wrong agent is used in this vaporizer. National and international standards are provided for by the keyed filler version of this vaporizer.
- 3. The Matrx VIP 3000 vaporizer must be secured in the upright position before it is connected to a patient. Excess dosage may be delivered if the vaporizer is moved suddenly during use.
- 4. When the Matrx VIP 3000 calibrated vaporizer is filled with anesthetic agent, the dial control must be set at "OFF", also when not in use and/or during transportation. The vaporizer must be secured in the upright position for a minimum of 10 minutes before it is connected to a patient via a breathing system. Excess dosage may be delivered if adequate time is not allowed for the liquid agent to return to its normal level. If the vaporizer has been transported with the dial control set at any position other than "OFF", or was transported other than in an upright position, it must be flushed with oxygen at 4 liters/minute for a minimum of 2 minutes to prevent incorrect dosage. If the vaporizer has been dropped, it MUST be returned to a Midmark authorized Service Center for functional checks.

- 5. Anesthetic agents are drugs; to prevent the hazard of prolonged inhalation of trace concentrations from the atmosphere, great care must be taken to avoid spilling of any agent during filling or draining. Patient exhaled anesthetic gases should be extracted from the operating theater by an approved anesthetic gas scavenging system.
- 6. The dial control MUST be set at "OFF" during all filling and draining operations. To prevent overfilling, the vaporizer MUST be secured in the upright position during the filling process. The filler Screw Cap must be tightened during use; the delivered concentration will be incorrect if the filler port is open during use.
- The Matrx VIP 3000 calibrated vaporizer may cease to function correctly if it is exposed to excessive temperature. Always store the vaporizer at temperatures between -5°F and 122°F (-20°and 50°C).
- The output of the Matrx VIP 3000 calibrated vaporizer is sensitive to barometric pressure. It
 may be necessary to use a correction factor when analyzing the output, especially at high
 altitudes. The barometric pressure is not normally of clinical importance.
- Anesthetic agents must be treated as pharmaceutical products; to avoid contamination, liquid agent must NEVER be drained into an open container and/or re-used. The liquid must ALWAYS be disposed of as a hazardous chemical.
- The Matrx VIP 3000 calibrated vaporizer must NEVER be modified, dismantled, calibrated or serviced by unauthorized personnel. The calibrated vaporizer MUST be serviced at a Midmark authorized Service Center.
- 12. The Matrx VIP 3000 calibrated vaporizer MUST be connected so that the flow of the gas to the patient is as indicated by the arrows on the device. The delivered concentration will be incorrect if the flow is reversed.

The Matrx VIP 3000 calibrated vaporizer has a relatively high resistance and must not be incorporated in a breathing system downstream of the common gas outlet.
 Before use, ALL connections must be checked for leaks and functional tests MUST be performed as described in the anesthetic machine User Manual.

1.2 User Responsibility The Matrx VIP 3000 calibrated vaporizer must be checked periodically. It must be operated and maintained in accordance with the instructions provided. A known defective or damaged vaporizer must not be used. Components, which are damaged, worn, distorted, broken, contaminated or missing must be replaced immediately. Should such repair or replacement be necessary, Midmark recommends that a verbal or written request be made only to a Midmark authorized agency. The vaporizer must not be altered or modified in any way without prior written consent from Midmark. The user of this vaporizer shall have sole responsibility for any malfunction, which results either from alteration by anyone other than Midmark authorized personnel, or from improper use, incorrect maintenance, improper repair or damage.

1.3 Servicing

Only qualified service personnel must service the Matrx VIP 3000 calibrated vaporizer. The contents of this manual are not binding. If any significant differences are found between the product and this manual, please contact Midmark for further information.

To ensure that the Matrx VIP 3000 calibrated vaporizer functions correctly, it must be serviced at regular intervals at a Midmark authorized Service Center. Midmark recommends that the vaporizer should be serviced at intervals not exceeding 12 months.

Qualified service personnel and genuine spare parts must be used for all servicing and repair requirements. Midmark will otherwise not assume responsibility for the materials used, the work performed, or any possible consequences of the same.

2.0 Description 2.1 General


When communicating with Midmark, please quote the model and serial number of the vaporizer, along with the approximate year of purchase. If the unit is being returned for repair, indicate the nature of the fault or the work required to be undertaken. Midmark can be contacted by telephone by dialing 1-800-Midmark, (1-800-643-6275). Alternatively, use FAX 1-877-725-6495

WARNING: This manual and all its associated documentation must be studied thoroughly before any attempt is made to install, operate or maintain any part of the Matrx VIP 3000 calibrated vaporizer. Failure to do so may result in patient injury.

The Matrx VIP 3000 calibrated vaporizer is designed for "out of circuit" use in continuous flow techniques of inhalation anesthesia.

The Matrx VIP 3000 calibrated vaporizer is temperature and flow compensated so that its output remains relatively constant despite cooling due to the vaporization and variations in inlet flow.

Every Matrx VIP 3000 calibrated vaporizer is labeled to show the name of the anesthetic agent for which it is designed and calibrated.

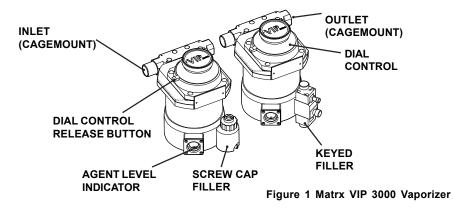

2.2 The Dial Control


To set the desired concentration of anesthetic agent, a single Dial Control with a concentration scale calibrated in % of anesthetic agent vapor per volume (v/v) is used. This Dial Control incorporates a release button (see Fig 1) to help prevent accidental rotation of the dial from the "OFF" to the "ON" position. A counter-clockwise rotation of the dial and simultaneous depression of the release button is required to set the vaporizer to the on position.





WARNING: Turn the vaporizer dial control to "OFF" when not in use


3.0 Specifications	Matrx VIP 3000 calibrated vaporizers are calibrated at 70°F (21°C). The variation in output with temperature, flow rate and duration of use is small, and the variation of output when used with ntermittent positive pressure ventilation is negligible.
3.1 Calibration	The Dial Control is calibratied to the levels shown on the performance curves in Section 4 of this manual.
3.2 Resistance to Gas Flow	.5 centimeters of Water (cm. H_2O) at the "OFF" setting at 5 liters/minute (LPM) O_2 .
3.3 Liquid Capacity	Amount of anesthetic agent to fully charge the vaporizer = 125 milliliters (ml.) (nominal).
3.4 Weight and Dimensions	Amount retained by wick system = 35 ml (nominal). Weight 15.2 lbs. (6.9 kg.) Height 8 in. (205 mm.) Depth 5.5 in. (140 mm.) Width 5.3 in. (135 mm.)
3.5 Temperature Range	The Matrx VIP 3000 calibrated vaporizer is designed to operate at temperatures between 59°F (15°C) and 96.8°F (36°C).


4.0 Performance 4.1 Performance Curves

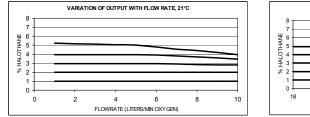

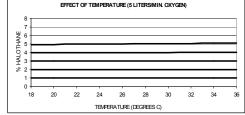

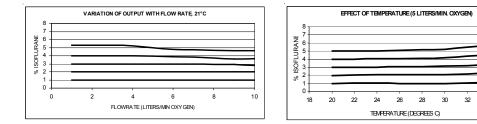

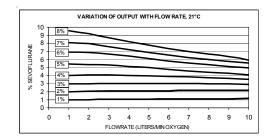

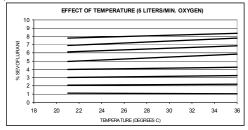

34 36

4.2 Effects of Variables 4.2.1 Anesthetic Consumption	The rate of consumption of anesthetic agent depends primarily on flow rate and vapor output concentration. As an approximate working figure, 1.0 milliliter of liquid agent is required to provide 200 milliliters of vapor. The approximate hourly consumption in milliliters of agent can be expressed as follows:
	3 X % X F, where % = the setting of the output percentage, F = input flow rate in liters/ minute.
	Example: if a vaporizer is set to deliver $2\% @ 6$ liters/minute total input gas flow rate: approximate rate of agent consumption = $3 \times 2 \times 6 = 36$ milliliters/hour.
	The figures are intended only for clinical guidance and are approximate. They may vary depending on the type of anesthetic agent used, accuracy of graduation of flow meters, etc., and will vary greatly if the vaporizer filler/drain port is not fully closed.
4.2.2 Temperature	The effects of variation in temperature are normally negligible at commonly used combinations of dial setting and ambient temperature.
	The Matrx VIP 3000 calibrated vaporizer responds very slowly to changes in ambient temperature and, as a safety feature, the temperature sensitive valve does not respond to temperatures below the range of approximately 53.5°F (12°C) to 59°F (15°C), thus preventing the valve from closing completely.
	Should the Matrx VIP 3000 calibrated vaporizer temperature fall lower than this, the output can be expected to be lower than that indicated on the dial control.
	At temperatures above the range shown on the performance curves, the Matrx VIP 3000 calibrated vaporizer output may be unpredictably high, particularly if the temperature approaches the boiling point of the anesthetic agent.

To avoid inaccuracies due to extreme temperatures, the Matrx VIP 3000 calibrated vaporizer should be allowed to attain a temperature in the range shown on the performance curves prior to use.

4.2.3 Barometric Pressure	The dial control is graduated in v/v percentage at 760 millimeters of mercury (mm-Hg). If the ambient pressure changes, the v/v% changes so that at an ambient pressure (P) mm-Hg, the delivered percentage (D) =
	Equation 1: D = $(\% \times 760) / P$; where % is the nominal setting of the vaporizer.
	It is generally accepted that the depth of anesthesia depends on the inspired partial pressure of agent and not the concentration by volume of agent.
4.2.4 Back Pressure (Steady)	To obtain a consistent depth of anesthesia when gross changes of barometric pressure occur, the v/v% must be changed in inverse proportion to the barometric pressure. The Matrx VIP 3000 calibrated vaporizer automatically makes this change, thus the effects of change in barometric pressure can be ignored for practical clinical purposes.
4.2.4.1 Low and Moderate Pressure	The Matrx VIP 3000 calibrated vaporizer cannot distinguish between pressures at the outlet due to barometric pressure and pressures in excess of barometric pressure which are due to steady back pressure applied by downstream components. Equation 1 above therefore applies with P now being the absolute pressure at the outlet, i.e. barometric pressure plus backpressure. Steady backpressure reduces the v/v percentage.
	Currently, it is unlikely that the steady backpressure imposed by commonly used downstream components (other than some ventilators) will exceed 30 mm-Hg at commonly used flow rates. Back pressures as high as this would reduce the delivered v/v percentage at 760 mm-Hg barometric pressure to:
	Equation 2: 760/790 = 0.96 of what would otherwise be expected. Under normal clinical circumstances, effects of this degree can be ignored.
4.2.4.2 High Pressure	Pressures, in excess of approximately 400 mm-Hg, should not be imposed on the Matrx VIP 3000 calibrated vaporizer, since this may overcome the loads imposed by internal thrust springs.

4.2.5 Back Pressure (Fluctuating)	Fluctuating backpressure may be imposed on the Matrx VIP 3000 calibrated vaporizer by downstream components and assisted or controlled ventilation to the patient. This can affect the vaporizer and increase the concentration by intermittently altering the pressures, and hence the flow distribution within the vaporizer. The greatest effects are observed at combinations of very low flow rates and low dial settings with large and rapid pressure fluctuations. This becomes progressively unimportant as the dial setting and flow rate increase and the magnitude and rate of cycling of the pressure fluctuations decrease.
	In clinical use, Matrx VIP 3000 calibrated vaporizers are considered unaffected by all fluctuating backpressures which would occur under all normal clinically encountered conditions relating to anesthesia.
4.2.6 Carrier Gas Composition	Small effects can occur when the carrier gas composition is changed from oxygen to either an oxygen/air or a nitrous oxide/oxygen mixture. As a general rule, variation of output with carrier gas composition can be considered of negligible clinical significance, since any effects are normally less than 10% of setting. Where changes do occur, the usual effect is that the output is slightly depressed when nitrous oxide is employed, compared to the output when only oxygen is the carrier gas. The presence of nitrous oxide reduces the required inspired concentration of volatile agent and this mitigates this small depression in output.
4.2.7 Time Out Of Service	If the anesthetic machine on which the Matrx VIP 3000 calibrated vaporizer is fitted is left for a period of time with no gases flowing, sensitive analyzers may detect small concentrations of agent at the anesthetic machine's outlet when the gas flow is turned "ON" and the vaporizer is set to "OFF". This concentration can be expected to decrease rapidly to zero within approximately 15 seconds at flow of 5 LPM, for example. This phenomenon is a normal characteristic of anesthetic vaporizers and anesthetic machines. It is considered to be clinically insignificant because of the small volume of vapor involved.
4.2.8 Other Variables	Ambient temperature, input flow rate and duration can often affect delivered concentrations, particularly when the vaporizer is used at extremes of the usual clinical range. The valve design and temperature compensation system of the Matrx VIP 3000 calibrated vaporizer reduces the effects to levels where, under most clinical conditions, their effect on vaporizer performance is not clinically significant. The nominal performance characteristics should be consulted for further details.

5.0 Principal of Operation

5.1 Vaporizer Sump and Valve Assembly

When in the "OFF" position, the rotary valve (within the dial control) makes a direct link between the inlet and outlet of the vaporizer. When the dial control is turned to on, the carrier gas is split into two streams, respectively designated "bypass" and "vaporizing chamber flow".

The vaporizing chamber is lined with two concentric wicks, which enclose a nickel-plated copper alloy helix, so that the space is converted into a long spiral outlet channel. The wicks are in contact with the liquid agent, thereby ensuring that the vapor is maintained at saturation concentration in the gas leaving the vaporizing chamber.

The amount of anesthetic agent picked up in the vaporizing stream can vary, due either to variation in room temperature or the cooling which takes place as the agent is vaporized. Each variation causes changes in the effective vapor pressures of the anesthetic agent, therefore, unless some form of compensation device is used, the output of the vaporizer for any given flow and dial setting would change with changes in temperature. The Matrx VIP 3000 calibrated vaporizer incorporates a temperature-compensating device (thermostat) that utilizes a bi-metallic strip that deflects according to temperature in order to control the proportion of carrier gas entering the vaporizing chamber.

If the temperature of the Matrx VIP 3000 calibrated vaporizer decreases, the thermostat closes and more carrier gas is admitted into the vaporizing chamber. If the temperature of the vaporizer increases, the thermostat opens and less carrier gas is admitted into the vaporizing chamber. In this way, the output of the vaporizer remains constant under conditions of changing temperature within the range specified.

Key to Figure 2.

- A Rotary Valve Inlet
- B Rotary Valve Outlet
- C Inlet to Bypass Chamber (Thermostat)
- D Vapor Chamber Inlet
- E Outlet Vapor Chamber to Calibrated Channel
- F Outlet Calibrated Channel to Vapor Outlet

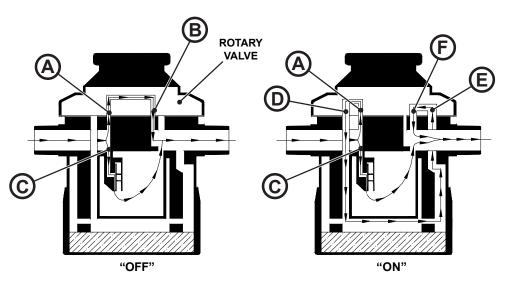


Figure 2 Schematic – Principle of Operation

When the dial control is set at "OFF", fresh gas enters the central chamber A and leaves via outlets B and C.

When the dial control is rotated to the on position, the bypass outlet B is occluded. Fresh gas enters and is split to the rotary valve inlet A and the inlet bypass chamber C. Bypass gas passes through C to the temperature compensated bypass chamber. The other gas stream passes through the vapor chamber inlet D and is ducted down the inside of the inner wick support to the base of the vaporizing chamber.

From the base of the vaporizing chamber, the gas passes up the helix wick support, which is sandwiched between the inner and outer wicks, thus giving a long gas pathway.

The saturated vapor leaves the chamber outlet via the outlet E to the outer channel. This annular channel forms a variable resistance pathway, which controls the concentration of vapor. The channel is of uniform width throughout its length, but with an increase in depth from beginning to end.

The saturated vapor leaves the channel at the outlet F and passes to the vaporizer outlet to mix with the bypass gas to form the final outlet concentration. To increase concentration, the dial is rotated counter-clockwise so that inlet E and outlet F move relatively down to a deeper part of the calibrated channel so that the flow resistance between E and F falls. It will be seen that counter-clockwise rotation of the dial control (rotary valve) results in a lower vapor chamber resistance, thus a greater proportion of the total gas flow passes through the vaporizing chamber, which results in a rise in outlet concentration.

As the Matrx VIP 3000 calibrated vaporizer has a long inlet pathway, this, coupled with similar volumes of bypass vapor and a small volume vaporizing chamber, means that it is not readily susceptible to a pumping effect. The vaporizer's high resistance means that it can be used ONLY BEFORE the patient circuit.

Should the temperature fall, the thermostat moves toward closed and more gas is diverted into the vaporizing chamber.

6.0 Installation



Warning: Keep the Matrx VIP 3000 calibrated vaporizer upright at all time. Do not carry the vaporizer by holding the dial control.



Warning: To help minimize administering multiple agent anesthesia to the patient, only one Matrx VIP 3000 calibrated vaporizer should be fitted to an anesthesia machine at any one time.

The Matrx VIP 3000 calibrated vaporizer must always be mounted between the flow-metering unit and the patient breathing circuit, but upstream of any absorber or humidifier.

Check the integrity of the fittings to ensure that they are leak tight. If in doubt, seek advice from the manufacturer of the equipment to which the vaporizer is attached.

Unless otherwise specified, all Matrx VIP 3000 calibrated vaporizers are supplied as standard with 23mm cagemount fitting inlet and outlet ports.

6.1 Mounting the Vaporizer

Cagemount fitted vaporizers normally have the standard 23-mm tapered ports: male (inlet) on the left and female (outlet) on the right when viewed from the front. There are two threaded holes at the rear of the vaporizer, which are utilized to secure the vaporizer onto the anesthesia machine using appropriate studs, spacers, and nuts.

- 1. Lightly coat the tapers with oxygen-safe grease such as Krytox.
- 2. Push the tapered fittings fully onto the appropriate tapers.



Warning: Ensure that all connections are gas tight before using the machine.



Warning: Before use, ALL connections must be checked for leaks and functional tests must be performed as described in the anesthetic machine user manual.



Warning: As there is no interlock fitted to the Matrx VIP 3000 calibrated vaporizers, only one vaporizer should be connected at any one time, thus assisting with the prevention of administering multiple agent anesthesia to the patient.

7.0 Operating Instructions



Warning: Ensure that the Matrx

VIP 3000 calibrated vaporizer is

Warning: Do not carry the Matrx VIP

3000 calibrated vaporizer by holding

To turn the vaporizer "ON" depress the dial control release button and turn the dial in a counter-clockwise direction.

To avoid inadvertent delivery of small

must be turned to "OFF" when the

vaporizer is not in use.

concentrations of agent, the dial control

upright at all times.

the dial control.


7.1 Turning Vaporizer "ON

7.2 Filling and Draining 7.2.1 General





Warning: The Matrx VIP 3000 calibrated vaporizer must be filled and used in an upright position.

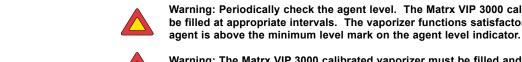

Warning: Periodically check the agent level. The Matrx VIP 3000 calibrated vaporizer must be filled at appropriate intervals. The vaporizer functions satisfactorily as long as the



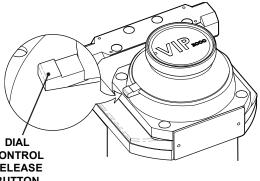


Figure 3 Turning the Vaporizer "ON "

Warning: Do not fill the vaporizer with any agent other than the one specified on the front label. The vaporizer is designed for that agent only. Use of any agent other than the one specified can be dangerous to the patient.

Warning: Do not fill the vaporizer unless the dial control is in the "OFF" position.

Warning: Do not turn the dial "ON" during filling, or attempt to fill the vaporizer beyond the full mark.

Warning: Do not drain the agent into any container other than a properly marked container.

VIP 3000 Calibrated Vaporizer O & M Manual Page 14

7.2.2 Screw Cap Filler 7.2.2.1 Filling Procedure

Caution: The vaporizer may be pressurized. Turn the screw cap slowly when filling or draining vaporizers which are filled with screw cap fillers.

1. Ensure that the Dial is in the "OFF" position. Remove the screw cap by turning it counter-clockwise.


2. Verify the agent to be used is the same as that specified on the front of the vaporizer. Pour the agent slowly into the filler opening, observing the agent level through the agent level indicator.

Note: If the vaporizer was dry before filling, the level will decrease slightly as the wicks absorb the agent.

3. When the agent level reaches the maximum level mark on the agent level indicator, the vaporizer is full. Replace the screw cap by turning it clockwise. To prevent leakage, ensure that the screw cap is fully tightened.

Warning: After draining the Matrx VIP 3000 calibrated vaporizer, fully tighten the drain plug before replacing the screw cap.

Remove the screw cap to reveal the drain plug. Invert the screw cap and use the slot in the top to unscrew the drain plug. Loosen the drain plug. Drain the agent into a properly marked container for disposal.

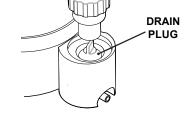

FILLING - TURN SCREW CAP

COUNTER-CLOCKWISE TO

REMOVE

SCREW

Figure 4 Filling – Screw Cap Filler

DRAINING - INVERT SCREW CAP AND TURN COUNTER- CLOCKWISE

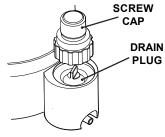


Figure 5 Draining – Screw Cap Filler



7.2.2.2 Draining Procedure

7.2.3 Keyed Filler 7.2.3.1 Filling Procedure

This filling system consists of three keyed elements, as follows: The anesthetic agent bottle collar, bottle adapter and the filing/draining unit fitted to the vaporizer.

1. Remove the cap and seal from the anesthetic bottle. Check that the bottle neck is not chipped or damaged. Fit the keyways of the bottle adapter to the keys of the bottle collar. Screw them together until fully tightened. The bottle is then ready for filling the vaporizer.



Note: Only the correct agent-specific adapter can be fitted into the matching filler socket.


Caution: The vaporizer may be pressurized. Turn the top retaining screw slowly when removing the dummy filler plug on vaporizers fitted with keyed fillers.

2. Ensure that the dial control is set to the "OFF" position. Turn the top retaining screw on the filler unit counter-clockwise and withdraw the dummy filler plug.

3. Hold the bottle upright below the filler socket and bend the adapter so that its end is horizontal and the two holes in the adapter are facing downwards. Insert the adapter into the filler socket.

4. After insertion, turn the top retaining screw clockwise to tighten it and seal the filler adapter in the filler socket.

5. Raise the bottle above the level of the filler socket, avoiding kinking the adapter tube. A steady stream of bubbles should emerge from the adapter inner tube within two seconds. If this does not occur, remove the bottle and adapter from the vaporizer and remove the adapter from the bottle. Carefully shake the adapter two or three times to clear the tube, then repeat instructions from Section 7.2.3.

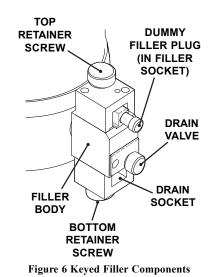

VIP 3000 Calibrated Vaporizer O & M Manual Page 16

6. When the vaporizer is filled to the maximum level mark in the agent level indicator, lower the bottle below the level of the filler socket and wait for five seconds to allow any agent in the adapter to drain back into the bottle, then unscrew the top retaining screw and remove the adapter from the filler. If there is any excess liquid agent, allow this to escape from the filler socket completely, then insert and fully tighten the dummy filler plug to prevent gas from escaping through the filler.

The Matrx VIP 3000 calibrated vaporizer is now ready for use.



Note: If the vaporizer was dry before filling, the level will decrease slightly as the wicks absorb the agent.

7.2.3.2 Draining Procedure

1. Fit the bottle adapter to an empty bottle. Insert the bottle adapter with its two holes facing upwards into the drain socket. Tighten the bottom retaining screw.



Note: Only the correct agent-specific adapter can be fitted into the matching drain socket.



Caution: The vaporizer may be pressurized. Unscrew the top retaining screw slowly.

2. Ensure that the dial control is in the "OFF" position. Ensure that the bottle is below the level of the drain socket and the tube is not kinked. For draining purposes, and to allow air to vent, unscrew the top retaining screw and remove the dummy filler plug from the filler socket.

3. Open the drain valve by turning it counter-clockwise. Allow the vaporizer to drain.

4. If it is not possible to complete the draining process, close the drain valve, loosen the bottom retaining screw, then remove the bottle and adapter from the vaporizer and remove the adapter from the bottle. Carefully shake the adapter two or three times to clear the tube, then reassemble and repeat instructions from paragraph 1.

5. When draining is complete, close the drain valve (clockwise), loosen the bottom retaining screw and remove the bottle and adapter. Replace the dummy filler plug into the filer socket and fully tighten the top retaining screw in a clockwise direction.

8.0 Checking the Calibration	The performance of the Matrx VIP 3000 calibrated vaporizers, which are in clinical use, is monitored by observing patient signs and consumption of anesthetic agent. Some users may, however, wish to employ analyzers to determine whether any abnormalities of performance have developed.
8.1 General	The following points must be considered when any measurements are being carried out on a Matrx VIP 3000 calibrated vaporizer to determine whether any abnormalities of performance have developed.
	1. In order to predict the concentration that the Matrx VIP 3000 calibrated vaporizer can be expected to deliver, the detailed nominal performance data and the preceding comments should be noted.
	2. The method of testing used should be representative of normal conditions of use.
	3. Any sampling techniques should be such as to ensure the following:
	a. The sample is fully representative of the vaporizer output, which may not be a homogeneous mixture at the vaporizer outlet.
	b. Absorption of agent by any connecting tubing is negligible.
	4. If a number of vaporizers are being examined simultaneously, the probability of them all being consistently in error is so remote as to be negligible and the cause of any apparent error is likely to lie in the employed method of testing.
	5. Consistent and reproducible analytical techniques should be used.
	6. If unexpected results are obtained, it is a wise precaution to repeat the observations, since the Matrx VIP 3000 calibrated vaporizer may be more reliable than the techniques used to observe its performance.
	 If unexpected results occur, it is also worthwhile to check for sources of error, e.g. flow meter leaks, absorption by adjacent components, etc. Full account should be taken of any extraneous effects on the analyzer, which may arise from changes in carrier gas composition.

9. If the anesthetic machine on which the Matrx VIP 3000 calibrated vaporizer is fitted is left for a period of time with no gases flowing, sensitive analyzers may detect small concentrations of agent for a short time at the machine outlet after the machine is turned "ON" with the vaporizer turned "OFF". This concentration can be expected to decrease rapidly to zero within approximately 15 seconds at 5 liters per minute. This phenomenon is a normal characteristic of anesthetic vaporizers and anesthetic machines.

10. At the zero or "OFF"-setting (where marked), it is not abnormal for small steady concentrations to be observed when using sensitive analyzers.

For field checking of the state of calibration, many techniques and analyzers are available. Midmark does not recommend any one technique or analyzer, but account must be taken of errors of use and calibration of analyzers. The reliability of both must be realistically considered.

The following method of checking may be used where special equipment is not available and a secondary check on analyzers is desirable. The characteristics of the vaporizer are such that if the vaporizer is satisfactory at one dial setting it should be satisfactory at all graduation settings.



Warning: Appropriate measures to handle exhaust gases and spillage should be carried out during this test.

- Ensure that the vaporizer is full and has remained at an ambient temperature of 69.8°F (21°C) for a minimum period of 3 hours.
- 2. With the vaporizer securely mounted, open the drain valve until no more liquid agent runs out. Close the drain valve.
- 3. With the dial set at "OFF", carefully and quickly refill the vaporizer with a measured amount of agent (approximately 70 milliliters) without spilling. Close the screw cap securely.
- 4. Ensure that the temperature has stabilized. Allow the vaporizer to remain at 21°C for one hour.

8.2 Analytical Techniques

- 5. Set the flow rate to 4 liters per minute O₂.
- 6. Note the time, turn the dial to 2% and check that the flow rate remains at 4 liters per minute, readjust if necessary.
- 7. Leave the vaporizer at this setting for 30 minutes, periodically checking and adjusting the flow rate if necessary.
- 8. Drain as described in Section 7.0. Measure the amount of liquid drained from the vaporizer.
- 9. The amount of liquid consumed should be as follows:
 - a. Halothane 10.5 milliliters
 - b. Isoflurane 12.3 milliliters
 - c. Sevoflurane 13.2 milliliters

It should be appreciated that the preceding method of checking is designed to be quick and easy under ordinary hospital conditions and that the method is somewhat imprecise. Nevertheless, it would be unusual for measured liquid agent consumption to vary by more than approximately 25% from the amounts shown above.

9.0 Maintenance



Warning: Do not modify, tamper with, or disassemble the Matrx VIP 3000 calibrated vaporizer. If the vaporizer is modified in any way there could be possible danger of damaging the unit and altering the accuracy of output concentration.

Warning: Do not immerse the Matrx VIP 3000 calibrated vaporizer in any liquid, including water.



Warning: Do not sterilize the Matrx VIP 3000 calibrated vaporizer.

Observation of the instructions provided, regular servicing and normal professional vigilance is normally all that is required to maintain the Matrx VIP 3000 calibrated vaporizer in a safe working condition.

9.1 Schedule

Every Two Weeks

When the level is low, the contents of the Matrx VIP 3000 calibrated vaporizer should be drained into an appropriately marked container. Discard the agent in a locally approved manner. Agents such as Isoflurane and Sevoflurane require less frequent intervals since they do not contain any additives or stabilizing agents.

Annually

The Matrx VIP 3000 calibrated vaporizer should be serviced annually at a Midmark authorized Service Center.

This service includes the following:

- 1. Complete disassembly of the vaporizer and its components.
- 2. Thorough cleaning.
- 3. Inspection for damage and wear.
- 4. Renewal of wicks, seals and any damaged, worn or outdated components.
- 5. Lubrication where necessary.
- 6. Checking the delivered vapor concentration under closely defined conditions at different temperatures. Re-graduation and adjustment where necessary.

9.2 Cleaning



Warning: Do not put water or another solvent in the Matrx VIP 3000 calibrated vaporizer. A vaporizer should be filled with the specified anesthetic agent only.

Clean the exterior of the Matrx VIP 3000 calibrated vaporizer with a damp cloth.

Never allow cleaning agents to accumulate in the filler or gas inlet and outlet ports, or around the dial control.

9.3 Contamination If any liquid other than the correct agent is put into the Matrx VIP 3000 calibrated vaporizer:

- 1. Drain and discard all the liquid.
- 2. Set the dial control to maximum concentration and flush the vaporizer with 4 liters per minute oxygen until no trace of the contaminant can be detected.
- 3. Allow 2 hours to elapse for the vaporizer temperature to stabilize. Test the vaporizer to ensure that no trace of contaminant is present.

If the contaminant is not volatile (e.g. water), drain the vaporizer and return it to a Midmark authorized Service Center.

9.4 Repairs

Repairs should only be carried out by Midmark authorized service representatives or agents.

10.0 Warranty

Such warranties are extended only with respect to the purchase of this product direct from Midmark or Midmark authorized dealers as new merchandise and are extended to the first buyer thereof, other than for the purpose of resale.

For a period of three (3) years from the date of original delivery to the first buyer or to buyers order, this product is warranted against functional defects in materials and workmanship and to conform to the description of the product contained in the Operation and Maintenance Manual and accompanying labels and inserts, provided that the same is operated under conditions of normal use, that regular periodic maintenance is performed and that replacements and repairs are made in accordance with the instructions provided.

The foregoing warranties shall not apply if the product has been repaired or serviced other than by Midmark or Midmark authorized service facilities, or other than in accordance with the written instructions provided by Midmark, or altered by anyone other than Midmark or Midmark authorized service facilities, or if the product has been subject to abuse, misuse, negligence or accident.

Midmark 's sole and exclusive obligation and the buyer's sole an exclusive remedy under the above warranties are limited to repairing or replacing, free of charge, at Midmark's option, a product which is confirmed as being defective by Midmark following the buyer's notification to Midmark in accordance with the instructions contained in the Servicing Section of the Operation and Maintenance Manual, not later than seven (7) days after the expiration date of the applicable warranty. Midmark shall not be otherwise liable for any damages including but not limited to incidental damages, consequential damages or special damages.

There are no express or implied warranties, which extend beyond the warranties herein above, set forth. Midmark makes no warranty of merchantability or fitness for a particular purpose with respect to the product or parts thereof.

Midmark Corporation For contact information, go to: www.midmark.com

© Midmark Corporation 2009



Because we care.